

solutions™

DEVELOPED BY BRASSELER USA

DIALITE® HP INTRA-ORAL PORCELAIN POLISHING SYSTEM



B-3738B-REV2-5022162L0-04.15

©2015 Brasseler USA. All rights reserved. Made in the U.S.A.

Order Number:

K0193



Figure Number	W17DCHP	W16DCHP	W18DCHP	W17DMHP	W16DMHP	W18DMHP
Head Size (1/10 mm)	060	040	100	060	040	100
Head Length (mm)	7.5	10.0	2.5	7.5	10.0	2.5












Figure Number	W17DHP	W16DHP	W18DHP
Head Size (1/10 mm)	060	040	100
Head Length (mm)	7.5	10.0	2.5

Only Available Direct from your Brasseler Representative
To order call 800.841.4522 or fax 888.610.1937
In Canada call 800.363.3838 or fax 866.330.4454
Visit our website at BrasselerUSA.com



K0193 DIALITE® HP INTRA-ORAL PORCELAIN POLISHING SYSTEM

	Order #	Head Size	Grit	Usage
	W16DCHP-RA	040	C	Polisher - Occlusal Surfaces
	W17DCHP-RA	060	C	Polisher - Labial/Lingual/Buccal
	W18DCHP-RA	100	C	Polisher - Interproximal Surfaces
	W16DMHP-RA	040	M	Polisher - Occlusal Surfaces
	W17DMHP-RA	060	M	Polisher - Labial/Lingual/Buccal

	Order #	Head Size	Grit	Usage
	W18DMHP-RA	100	M	Polisher - Interproximal Surfaces
	W16DHP-RA	040	F	Polisher - Occlusal Surfaces
	W17DHP-RA	060	F	Polisher - Labial/Lingual/Buccal
	W18DHP-RA	100	F	Polisher - Interproximal Surfaces

Tips & Techniques:

Complimentary items:

Power Suggestion

Sterilization:

Recommended Speeds:

Polishers produce best results when using all three steps in order (coarse, medium and fine); Dialite is best when used on feldspathic porcelain.

Intra-Oral LD Grinders (LD11M.RA or LD11C.RA) and Dialite Diamonds (Round, Football and Round-End Taper)

Brasseler Forza F1 Electric Attachment with light pressure

Autoclave only

5,000-8,000 rpms



BRASSLER
USA®