

solutions™

DEVELOPED BY BRASSELER USA

DIALITE® HP LAB SYSTEM



B-3742B-REV2-5022174L0-05.15

©2015 Brasseler USA. All rights reserved. Made in the U.S.A.

Order Number:

K0174

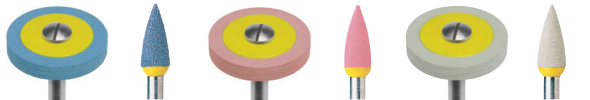


Figure Number	R17DCHP	H2DCHP	R17DMHP	H2DMHP	R17DFHP	H2DFHP
Head Size (1/10 mm)	170	040	170	040	170	040
Head Length (mm)	2.5	13.0	2.5	13.0	2.5	13.0


Only Available Direct from your Brasseler Representative
To order call 800.841.4522 or fax 888.610.1937
In Canada call 800.363.3838 or fax 866.330.4454
Visit our website at BrasselerUSA.com




BRASSELER
USA®

solutions™

K0174 DIALITE® HP LAB SYSTEM

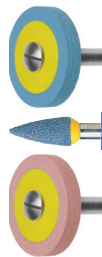

Order #	Head Size	Grit	Usage
R17DCHP-HP	170	C	Polisher - Contact Surfaces
H2DCHP-HP	040	C	Polisher - Occlusal Surfaces
R17DMHP-HP	170	M	Polisher - Contact Surfaces



Order #	Head Size	Grit	Usage
H2DMHP-HP	040	M	Polisher - Occlusal Surfaces
R17DFHP-HP	170	F	Polisher - Contact Surfaces
H2DFHP-HP	040	F	Polisher - Occlusal Surfaces

TIPS & TECHNIQUES:

Polishers produce best results when using all three steps in order (coarse, medium and fine); Coarse blue Dialites for adjusting and polishing; medium pink for pre-polishing and fine gray for final polishing

Complimentary items:

Dialite HP Polishers, Dialite Extra-Oral Diamond Paste (5015241U0), Diamond Discs (911H, 911HH, 911HV)

Power Suggestion:

Brasseler Forza L50K Laboratory Motor with light pressure

Sterilization:

Autoclave only

Recommended Speeds:

8,000-10,000 rpms



BRASSLER
USA®