


# solutions™

DEVELOPED BY BRASSELER USA

## DIALITE® ZR INTRA-ORAL POLISHING SYSTEM FOR ZIRCONIA

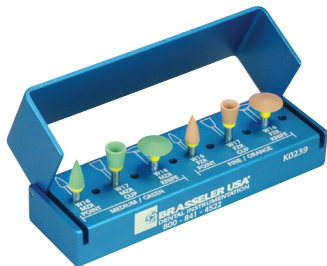

B-3794B-REV2-5022158L0-04.15  
©2015 Brasseler USA. All rights reserved. Made in the U.S.A.


Order Number:  
**K0239**




| Figure Number          | W16MZR.RA | W16FZR.RA | W17MZR.RA | W17FZR.RA | W18MZR.RA | W18FZR.RA |
|------------------------|-----------|-----------|-----------|-----------|-----------|-----------|
| Head Size<br>(1/10 mm) | 040       | 040       | 060       | 060       | 100       | 100       |
| Head Length<br>(mm)    | 10.0      | 10.0      | 7.5       | 7.5       | 2.5       | 2.5       |

Only Available Direct from your Brasseler Representative  
To order call 800.841.4522 or fax 888.610.1937  
In Canada call 800.363.3838 or fax 866.330.4454  
Visit our website at [BrasselerUSA.com](http://BrasselerUSA.com)



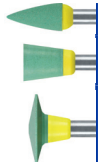

| Order #   | Head Size | Grit | Usage                             |
|-----------|-----------|------|-----------------------------------|
| W16MZR.RA | 040       | M    | Polisher - Occlusal Surfaces      |
| W17MZR.RA | 060       | M    | Polisher - Labial/Lingual/Buccal  |
| W18MZR.RA | 100       | M    | Polisher - Interproximal Surfaces |

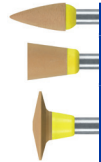

| Order #   | Head Size | Grit | Usage                             |
|-----------|-----------|------|-----------------------------------|
| W16FZR.RA | 040       | F    | Polisher - Occlusal Surfaces      |
| W17FZR.RA | 060       | F    | Polisher - Labial/Lingual/Buccal  |
| W18FZR.RA | 100       | F    | Polisher - Interproximal Surfaces |

#### TIPS & TECHNIQUES:

Polishers produce best results when using both grits in order (medium and fine); medium green for pre-polishing and fine orange for final polishing. Any adjustments should be made with the LD Grinder in coarse Green or Dialite Diamonds.

Complimentary items:

Intra-Oral LD Grinders (LD11M.RA or LD11C.RA) and Dialite Diamonds (Round, Football and Round-End Taper)

Power Suggestion:

Brasseler Forza F1 Electric Attachment with light pressure

Sterilization:

Autoclave only

Recommended Speeds:

5,000-8,000 rpms (reduce speed by half for final polish)



**BRASSLER**  
USA®