

HygienePro™ **PROPHYLAXIS** Glisten™ **PASTE**

CONTENTS:

HygienePro™ Glisten™ Prophylaxis Paste contains a unique blend of polishing and cleaning agents. **HygienePro™ Glisten™** Prophylaxis Paste is for professional use only and is designed for application during standard dental practice hygiene procedures. Contains fluoride at a concentration of 1.23% fluoride ion.

INDICATIONS:

For cleaning and polishing procedures as part of a professionally administered prophylaxis treatment. **NOTE:** Prophylaxis pastes containing fluoride are not considered a replacement for other forms of fluoride therapy.

PRECAUTIONS:

Excessive ingestion of fluoride can cause nausea. **HygienePro™ Glisten™** Prophylaxis Paste should be expectorated and **NOT SWALLOWED.**

SUGGESTED APPLICATIONS:

FINE: Gentle polishing and cleansing.

MEDIUM-FINE: Specially designed for plaque removal and polishing of children's teeth.

MEDIUM: Normal polishing, cleansing and plaque removal.

COARSE: Medium to heavy stain and plaque removal.

EXTRA COARSE: Heavy stain build-up and plaque removal.

ADMINISTRATION:

For use only by individuals who are professionally trained in the performance of dental prophylaxis.

DIRECTIONS FOR USE:

Each unit dose cup contains enough prophylaxis paste for one (1) prophylaxis procedure. Use any commercially available prophylaxis angle/rubber cup combination. To fill the cup press the cup face into the paste while running the angle at low speed. To avoid overheating the tooth surface, maintain sufficient paste in the cup throughout the procedure. Evacuate saliva to avoid swallowing and/or excessive dilution of the paste.

CAUTION:


U.S. Federal Law prohibits dispensing without prescription.

HOW SUPPLIED:

FLAVOR	Fine	Med	Coarse	Extra Coarse	Kids Med-Fine
Mint	X	X	X	X	
Cherry		X	X		
Root Beer		X	X		
Bubble Gum		X	X		
Chocolate		X	X		
Kids Asst.					X
Adult Asst.		X	X		

Brasseler USA®
One Brasseler Blvd.
Savannah, GA 31419
www.brasselerusa.com

MATERIAL SAFETY DATA

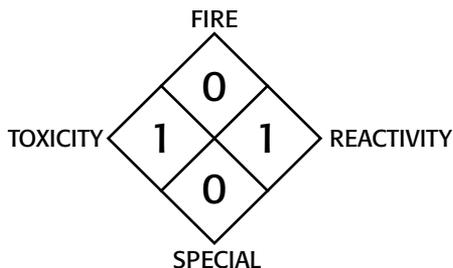

HAZARD RATING

4=EXTREME
3=HIGH
2=MODERATE
1=SLIGHT
0=INSIGNIFICANT

Manufacturer's Name: Brasseler USA®
Address: One Brasseler Blvd., Savannah, GA 31419
Telephone Number: 800-841-4522
Web Address: www.brasselerusa.com

Product Identification

Product Name:
HygienePro™ Glisten™ Prophylaxis
Paste With Fluoride

Hazardous Ingredients

Hazardous Components:
Fluoride Ion: Sodium Fluoride Active

Physical/Chemical Characteristics

DOES NOT APPLY

Fire and Explosion Hazard Data

DOES NOT APPLY

Reactivity Data

Stability: Stable
**Incompatibility
(Materials to Avoid):** N/A
**Hazardous Decomposition
of By-products:** N/A
Hazardous Polymerization: N/A
Conditions to Avoid: N/A

Health Hazard Data

Route(s) of Entry:
Inhalation? N/A
Skin? N/A
Ingestion? Yes
Health Hazards (Acute and Chronic):
N/A
Monographs? N/A
Signs and Symptoms of Exposure: N/A

Emergency and First Aid Procedures:

Eye: Flush with water. If irritation persists, contact physician.
Ingestion: Call a physician at once. Immediately give copious quantities of milk or water if milk is not available.

Precautions for Safe Handling and Use

Steps to be Taken in Case Material is Released or Spilled: N/A
Waste Disposal Method: N/A
Precautions to be Taken in Handling and Storing: N/A
Other Precautions: N/A

Control Measures

DOES NOT APPLY