




BRASSELER
USA®

FEATHER LITE™ POLISHERS BY BRASSELER USA

- ▶ Flexible Polishing Spirals
- ▶ Easily Adapts to Any Surface
- ▶ Creates an Enamel-Like Finish
- ▶ Preserves Surface Differentiation While Polishing

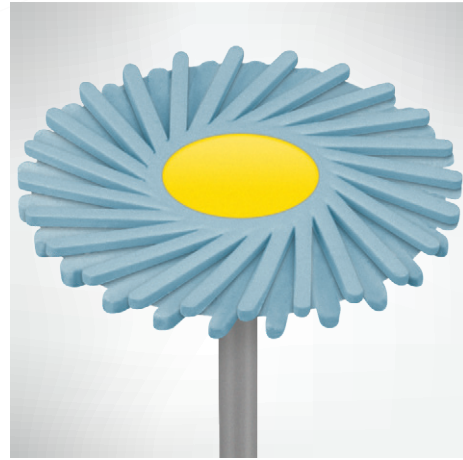

BY YOUR SIDE

Dialite® FEATHER LITE™

THREE-STEP PORCELAIN POLISHING SYSTEM

Brasseler USA introduces a revolutionary new addition to the Dialite family of porcelain polishers: Feather Lite. The Feather Lite polishing system consists of diamond-impregnated, polyurethane, multi-use polishers with unique flexible spirals designed to separate and “feather” out. This feathering helps maintain surface differentiation when polishing porcelain restorations, allowing the instrument to quickly adapt to any surface including occlusal, interproximal and contact areas. Available in a 26mm extra-oral head size and a 14mm intra-oral head size, both designs are offered in coarse, medium and fine grits. The new Dialite Feather Lite polisher will help save chair time as you quickly polish any ceramic restoration, including IPS e.max® and zirconia. Use Dialite Feather Lite as the final polish for all your crown and bridge restorations.


NEW DESIGN!

K0277

Dialite Feather Lite
Extra-Oral Kit



Dialite Feather Lite Intra-Oral Polishers

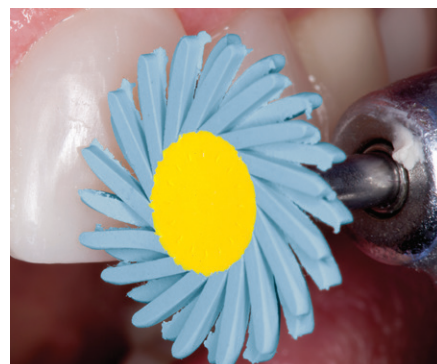

STEP 1



STEP 2



STEP 3

Photos courtesy of Douglas A. Terry, DDS



The revolutionary new Dialite Feather Lite is the first system to polish occlusal anatomy to a high brilliant luster efficiently and easily. The Feather Lite polishes grooves and elaborate occlusal structure effortlessly.

— John A. Sorensen, DMD, PhD

The Brasseler Dialite Feather Lite is by far the best polisher system I have ever used to give a natural finish to developmental and fine grooves on anterior teeth and occlusal anatomy on posterior teeth.

— Edward A. McLaren, DDS





TWO-STEP COMPOSITE POLISHING SYSTEM

Like the Dialite Feather Lite, DiaComp Feather Lite features the same flexible spiral design and versatility. The DiaComp Feather Lite is also a diamond-impregnated, reusable polishing system. However, this two-step procedure is specifically designed to polish any type of composite intra-orally. Available in medium and fine grits, this polisher will easily adapt to all tooth surfaces and quickly restore an enamel-like finish to your composite restorations.




Polishing composite and ceramic materials with Feather Lite polishers can improve the smoothness, color and translucency of any direct or indirect restoration, while enhancing the integration of the biomaterial to tooth structure. The flexible spirals allow the instrument to quickly adapt to any surface for an optimal polished surface.

— Douglas A. Terry, DDS



DiaComp Feather Lite Intra-Oral Polishers

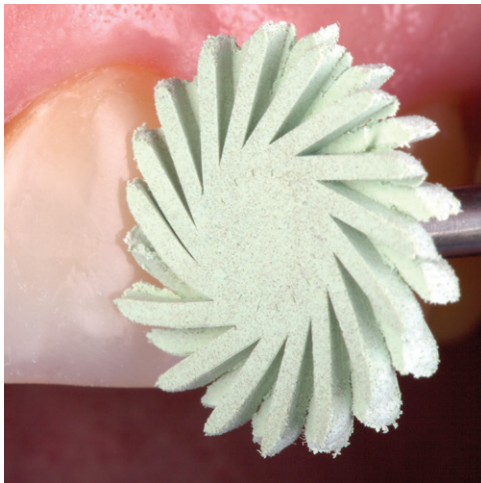

STEP 1



STEP 2

Photos courtesy of Douglas A. Terry, DDS

TIP: Following the final contour with ET® carbides or diamonds, use the DiaComp Feather Lite two-step polishers at slower speeds and with water to polish composites.

— Ronald E. Goldstein, DDS

All Feather Lite products should be run at low speeds and sterilized via autoclave in a bur block to maximize the life of the polisher. Intra-oral Feather Lite polishers work best when used with water.

► Dialite® Feather Lite™ Extra-Oral Polishers

NEW DESIGN!



Order No.	FL26CHP	FL26MHP	FL26FHP
Head Size (1/10 mm)	260	260	260
Cutting Length (mm)	1.6	1.6	1.6
Grit	Coarse	Medium	Fine

11 - HP - Straight Handpiece (44.5 mm)

Qty/Pack	1	1	1
----------	---	---	---

Sterilization: Autoclave
Each grit available in kit K0277

Recommended Speed: 4-10,000 RPM*
*for final polish reduce speed by half (2-5,000 RPM)

► Dialite® Feather Lite™ Intra-Oral Polishers



Order No.	FLW14C.RA	FLW14M.RA	FLW14F.RA	FLW14A.RA
Head Size (1/10 mm)	140	140	140	140
Cutting Length (mm)	1.6	1.6	1.6	1.6
Grit	Coarse	Medium	Fine	Coarse / Medium / Fine

21 - RA - Right-Angle Latch (22 mm)

Qty/Pack	3	3	3	6 (2 of each)
----------	---	---	---	---------------

Sterilization: Autoclave

Recommended Speed: 5-8,000 RPM*
*for final polish reduce speed by half (2-4,000 RPM)

► Diacomp™ Feather Lite™ Intra-Oral Polishers



Order No.	FLDC5M	FLDC5	FLDC5-5M
Head Size (1/10 mm)	140	140	140
Cutting Length (mm)	1.6	1.6	1.6
Grit	Medium	Fine	Medium / Fine

21 - RA - Right-Angle Latch (22 mm)

Qty/Pack	3	3	6 (3 of each)
----------	---	---	---------------

Sterilization: Autoclave

Recommended Speed: 5-8,000 RPM*
*for final polish reduce speed by half (2-4,000 RPM)



Visit our website at BrasselerUSA.com

To order call 800.841.4522 or fax 888.610.1937. In Canada call 800.363.3838

