

# solutions™

DEVELOPED BY BRASSELER USA

## DIALITE® ZR ADJUSTING, FINISHING & POLISHING SYSTEM FOR ZIRCONIA



B-3795B-REV2-5022164L0-04.15

©2015 Brasseler USA. All rights reserved. Made in the U.S.A.

Order Number:

**K0262**



Figure Number	W16MZR	W16FZR	W17MZR	W17FZR	W18MZR	W18FZR
Head Size (1/10 mm)	040	040	060	060	100	100
Head Length (mm)	10.0	10.0	7.5	7.5	2.5	2.5

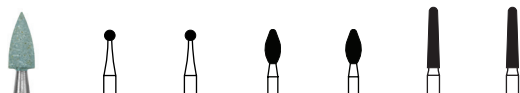















Figure Number	LD11CZR	8801LDF	801LDEF	8369DF	369DEF	8856DF	856DEF
Head Size (1/10 mm)	030	014	014	025	025	016	016
Head Length (mm)	7.5	—	—	5.5	5.5	8.0	8.0

Only Available Direct from your Brasseler Representative  
To order call 800.841.4522 or fax 888.610.1937  
In Canada call 800.363.3838 or fax 866.330.4454  
Visit our website at [BrasselerUSA.com](http://BrasselerUSA.com)



**BRASSELER**  
USA®

	Order #	Head Size	Grit	Usage
	W16MZR.RA	040	M	Polisher - Occlusal Surfaces
	W17MZR.RA	060	M	Polisher - Labial/Lingual/Buccal
	W18MZR.RA	100	M	Polisher - Interproximal Surfaces
	W16FZR.RA	040	F	Polisher - Occlusal Surfaces
	W17FZR.RA	060	F	Polisher - Labial/Lingual/Buccal
	W18FZR.RA	100	F	Polisher - Interproximal Surfaces

	Order #	Head Size	Grit	Usage
	LD11CZR.RA	030	C	Grinder - Adjusting
	8801LDF-FG	014	F	Endo Access/Small Adjustments
	801LDEF-FG	014	EF	Endo Access/Small Adjustments
	8369DF-FG	025	F	Occlusal Surfaces
	369DEF-FG	025	EF	Occlusal Surfaces
	8856DF-FG	016	F	Contact Surfaces/Crown Removal
	856DEF-FG	016	EF	Contact Surfaces/Crown Removal

### TIPS & TECHNIQUES:

Polishers produce best results when using both grits in order (medium and fine); medium green for pre-polishing and fine orange for final polishing. Any adjustments should be made with the LD Grinder in coarse Green and/or Dialite Diamonds.

### Complimentary items:

Intra-Oral LD Grinders (LD11M.RA or LD11C.RA) and Dialite Diamonds (Round, Football and Round-End Taper)

### Power Suggestion:

Brasseler Forza F1 Electric Attachment with light pressure

### Sterilization:

Autoclave only

### Recommended Speeds:

Polishers: 5,000 - 8,000 rpms (reduce speed by half for final polish)  
Diamonds: 60,000 rpms, use copious amounts of water