## HygienePro™ Ultrasonic Insert

### Directions for Use

Brasseler USA HygienePro Ultrasonic Inserts are designed to be used with magnetostrictive ultrasonic scaling devices by Dental Professionals only.

Brasseler USA HygienePro Ultrasonic Inserts are warranted against defects in materials or workmanship for 90 days from the date of purchase. This warranty is subject to the following conditions:

1. Warranty does not apply to inserts subject to improper use, abuse or improper cleaning or sterilization.
2. Warranty does not apply to inserts which have been bent or reshaped by the user.

Inserts with worn, bent, or altered tips should be disposed of to avoid possible breakage and/or patient injury.

### Returns

Once the security label on the insert's outer package has been broken, the product is considered "used" and cannot be returned for credit for any reason other than manufacturing defect.

To return product for credit due to manufacturing defect covered under warranty, please contact Brasseler USA's Customer Service Department at 1-800-841-4522 for a Return Material Authorization (RMA) number and return label. When product is returned, the Brasseler USA-assigned RMA number must accompany the product. Brasseler USA is not responsible for any returned product without an assigned RMA number.

During all scaling procedures, use the lowest effective power setting. Avoid contact of the activated tip with the lips, cheek and tongue. Adjust the water flow to deliver the appropriate amount of water to the tip. Using a light touch, adapt the tip so that it is parallel to the tooth surface, keeping the tip moving at all times.

**NOTE:** *The unit's operating frequency must match the frequency of the scaler insert used (either 25 kHz or 30 kHz).*

### Cleaning and Sterilization

Remove gross debris by placing the insert in an ultrasonic cleaner basket and immersing them in a non-ionic, non-ammoniated cleaning solution.

Remove inserts from the cleaner and dry completely with an air syringe or clean paper towel. Inserts may also be cleaned using a nylon bristle brush and detergent for debris removal, then rinse and dry completely.

Steam sterilize at temperatures not to exceed 275 degrees F (135 degrees C). Do not use cold sterilization, chemical disinfection, dry heat, or chemical vapor methods.

**NOTE:** *Lubricate the O-ring with water before placing the inserts into the handpiece. Depress the foot pedal to fill the handpiece with water and slowly turn the insert until it snaps into place. The O-ring may also be lubricated with a small amount of petroleum jelly.*

THE LIMITED WARRANTY STATED HEREIN IS THE SOLE AND EXCLUSIVE WARRANTY WITH RESPECT TO THE INSERT, AND BRASSELER USA HEREBY DISCLAIMS ANY AND ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING, WITHOUT LIMITATION, ANY STATUTORY WARRANTIES OR ANY WARRANTIES OF MERCHANTABILITY, FITNESS FOR USE OR FITNESS FOR ANY PARTICULAR PURPOSE.

BRASSELER USA WILL NOT BE RESPONSIBLE FOR ANY INCONVENIENCE, LOSS, INJURY, OR DIRECT, INDIRECT, TORT, EXEMPLARY, PUNITIVE, SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGE ARISING FROM THE POSSESSION OR USE OF THE INSERT.

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