



**BRASSELER**  
USA®

## **Dialite® Extra Oral Polishing Paste Instructions for Use**

### **Intended Use**

Dialite® Extra Oral Polishing Paste is a water-soluble diamond polishing compound available in various grits used to create a variety of surface finishes from matte to high lustre in ceramics, metals and composites.

### **Directions**

Choose a preferred rotary applicator tool (bristle brush, felt, cotton wheel) and run it through the paste in the jar at a speed of around 5,000 rpm - the friction will melt the paste onto the applicator tool. Do not remove paste from the tub or add water - there is no need.

Once the applicator tool has a layer of paste on the outside (edges) and starting at a slow speed (around 5,000 rpm), spread the paste onto the desired surface and begin polishing keeping the tool in contact with the surface. Speed up as needed. Within a few seconds the finish will begin to appear. There are several methods to polishing - side to side, front to back, pull stroke, but keep the applicator in contact with the surface.

Dialite® Extra Oral Polishing Paste is completely water soluble for easy cleanup.

### **Contraindications**

Used in the proper way as stated above, minimal waste is achieved, and no adverse effects should be realized.

### **Packaging**

Black jar.

### **Storage**

Store in a dry place preferably at room temperature. Keep away from extreme heat. If left in direct sunlight for a long period of time, paste may become slightly viscous and sticky. Left at room temperature, the material will revert to normal form. Always replace cap after use. Keep out of reach of children.

For use by dental professionals.

Glossary of Symbols: [Brasselerusadental.com/resources](https://brasselerusadental.com/resources)

Manufactured for  
Brasseler U.S.A. Dental, LLC  
One Brasseler Blvd.  
Savannah, GA 31419 USA  
Telephone 800-841-4522

Page 1  
2025-07-10  
IFU-PM500-EN  
REV1